

***HOOFix*® Abscess Kit**

- with the ***HOOFix*® Emergency Boot** that can be used for turnout & reused.

Kit Contents: Everything needed to treat a hoof abscess in 1 convenient package.

- ▶ *HOOFix*® Emergency Boot
- ▶ Epsom Salt Poultice Gel
- ▶ Epsom Salt Soak Bag
- ▶ Cohesive Flexible Bandage rolls (2)
- ▶ Felt Hoof Pads (4)
- ▶ Applicators (4)

Application Instructions

1. Pick out hoof. Brush off loose dirt and rinse hoof if possible.
2. Prepare 3-4 cups of very warm water.
3. Unwrap Epsom Salt Soak bag and place hoof in bag.
4. Add warm water to cover the hoof.
5. Wrap cohesive flexible bandage around the top of bag to secure it on the leg (being careful not to wrap too tight). Allow hoof to soak for approximately 10 minutes if possible.



Plum Shade Farm
www.PlumShadeFarm.com
103 Youngs Road, Coatesville, PA 19320
phone: 610.486.0708 fax: 610.444.2522

6. Open Epsom Salt Gel container and with wood applicator, apply a layer of gel to the felt pad. Use approximately 25% of the gel. In cold weather warming the gel makes it easier to spread (Note: If abscess is at coronary band, cut felt pad in half and apply gel.)



7. Remove the soak bag from the horse's hoof and apply the felt pad to the bottom of the hoof with the gel side touching the sole of the hoof.



8. Dry off hoof and secure the felt pad to the sole (or cut pad for coronary band) with cohesive flexible bandage. DO NOT wrap tightly around coronary band.



9. Place hoof in HOOFix® Emergency Boot and secure Velcro tabs. Poultice pad and HOOFix® Emergency Boot can be left on for several days before changing. Consult with your vet or farrier as needed.



Horses can be turned out while wearing boot.

HOOFix® - the brand you can trust

